ENGLISH

In this device we have integrated a radio transceiver to communicate wirelessly between device and carephone. In order for the device to send alarms through the carephone, you need to pair them.

How to pair the radio transceiver with the carephone

1. Connect the power to the base unit (CONNECT-IT).
2. Start the epilepsy alarm (companion mini): Press the recessed "TEST" button with a pen. The "ALARM" lamp starts flashing.
3. Start the base unit by pressing the recessed "PAIR" button with a pen. The "POWER" lamp starts to glow green. Wait until the "ALARM" light on the epilepsy alarm stops flashing.
4. Start pairing mode on the security alarm: See the security alarm manual for instructions.
5. Pair the epilepsy alarm: Press the recessed "TEST" button on the epilepsy alarm. The "ALARM" lamp starts flashing. Tap the device to send a test alarm. Wait for the confirmation tone from the security alarm.
6. When you hear the confirmation tone: press the alarm button on the security alarm to confirm connection of the epilepsy alarm. Another confirmation beep is heard.
7. When the "ALARM" light on the epilepsy alarm has stopped flashing, the configuration is complete.

For more information on pairing the carephone with the radio transceiver, refer to the manual of the carephone.

How to test the alarm after pairing

On the epilepsy sensor: Simulate movement by shaking the epilepsy alarm. Check that the alarm is received at the Alarm Receiving Centre.

This is how it works

1. When an alarm is triggered on the device, the radio transceiver sends the alarm to the carephone.
2. The carephone passes the alarm to the alarm receiving centre or another receiver.

Battery warning

The device automatically alarms at low battery. Replace the battery as soon as possible after low battery level has been indicated.

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